


# CAPRICORNIA NuTORIC® Soft Toric Lens

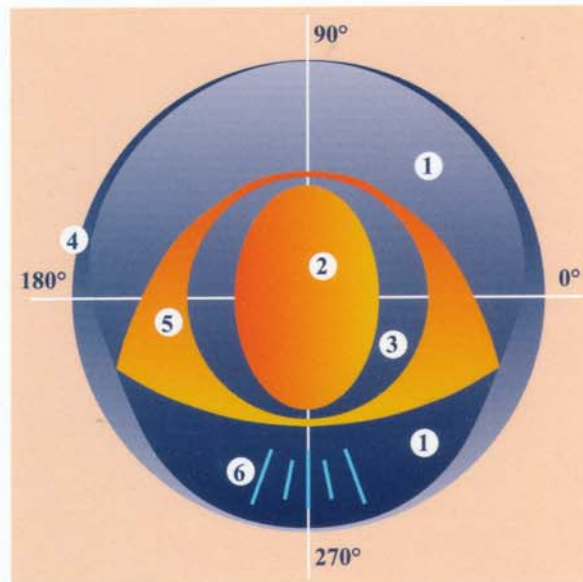

## Design Feature

## Benefit

Comfort chamfers (1)

Improved comfort and stability

Generated back surface toricity (2)

Greater reproducibility and stability

Prism-free optic zone (3)

No interference with binocular status; reduced thickness

Constant edge thickness (4)

Improved stability and comfort

Prism stabilisation zones (5)

Greater stability without excessive thickness

Laser orientation marks - 10° separation (6)

Easy assessment of lens rotation

- Patented design and winner of an Australian Design Award
- Automatic increase in stabilisation for difficult prescriptions
- Extra (X) and double extra (XX) stabilisation options
- Available with SA Multifocal technology for presbyopes

- Straightforward empirical fitting
- Choice of 4 materials and 7 tints
- Comprehensive Per Case warranty
- Full technical support and advice

# CAPRICORNIA NuTORIC® Soft Toric Lens

## Specifications

Materials	38%, 45G, 58%, 67%
Manufacture	Lathe-cut
Design	Front - spherical with comfort and stabilisation geometries Back - toric
Total Diameters	13.50 (0.50) 15.50mm
Base Curves	7.80 (0.20) 9.20mm
Spherical powers	to +/-25.00D
Cylindrical powers	-0.50 to -11.00D
Axes	Any
Orientation marks	5, at six o'clock and 10° intervals either side
Tints	7 cosmetic and handling tints in 3, 10, 20 and 30% intensities

## Fitting Guide

NuToric lenses are fitted empirically from vertexed spectacle Rx, HVID and keratometry readings.

### Base curve selection

Flat K (D)	46.36-45.05	45.05-43.79	43.79-42.62	42.62-41.49	41.49-40.42
Flat K (mm)	7.28-7.49	7.49-7.71	7.71-7.92	7.92-8.13	8.13-8.35
Indicated base curve	8.20	8.40	8.60	8.80	9.00

### Total diameter selection

Material	38	45G	58	67
Total diameter	HVID + 2.50mm	HVID + 2.50mm	HVID + 2.50mm	HVID + 3.00mm



**CAPRICORNIA  
CONTACT LENS**

From within Australia 1300 650 994  
 From within New Zealand 0800 777 118  
 From outside Australia 61 (7) 3208 8500  
 Email [info@capricorniacontactlens.com.au](mailto:info@capricorniacontactlens.com.au)  
 Web [www.capricorniacontactlens.com.au](http://www.capricorniacontactlens.com.au)