



CAPRICORNIA  
CONTACT LENS

## CAPRICORNIA MODCON RGP

The **Capricornia Modcon** is a versatile RGP lens that incorporates the latest thinking in lens design. It is recommended as an ideal first choice for most Rigid Lens patients.

**Modcon** features a unique tangential peripheral design and is a modern version of the original Conoid design, which increases comfort and tear circulation.

- Wide choice of materials
- Toric version available

### Fitting guidelines

Choose BOZR from the following table:

Amount of Corneal toricity	Initial BOZR (assumes 10.00mm TD)
0.0 to 0.15mm	on flat K
0.15 to 0.30mm	flat K-0.05mm
0.30 to 0.50mm	flat K-0.10mm
>0.50mm	toric design indicated

Choose Total Diameter as 1.50mm less than HVID to facilitate a lid attachment fit (larger lenses generally give better comfort and centration). If the upper lid is unusually high, decrease the indicated diameter by 0.50mm to achieve an interpalpebral fitting.

Lens Power calculated from vertexed spectacle Rx compensated for any tear lens effects.

Look for a minimal apical clearance fluorescein pattern surrounded by an even alignment pool and a 0.4mm wide band of edge clearance.

Centration should be central or slightly high.